## **REFERENCES**

- 1. Gad El-Rab MO; Kambal AM. Evaluation of a Brucella enzyme immunoassay test (ELISA) in comparison with bacteriological culture and agglutination. J Infect 1998; 36(2):197-201.
- Mikolon AB; Gardner IA; Hietala SK; Hernandez de Anda J; Chamizo Pestana E; Hennager SG; Edmondson AJ. Evaluation of North American antibody detection tests for diagnosis of brucellosis in goats. J Clin Microbiol 1998; 36(6):1716-22.
- Bowden RA; Cloeckaert A; Zygmunt MS; Bernard S; Dubray G. Surface exposure of outer membrane protein and lipopolysaccharide epitopes in Brucella species studied by enzymelinked immunosorbent assay and flow cytometry. Infect Immun 1995; 63(10):3945-52.
- Baldi PC; Miguel SE; Fossati CA; Wallach JC. Serological follow-up of human brucellosis by measuring IgG antibodies to lipopolysaccharide and cytoplasmic proteins of Brucella species. Clin Infect Dis 1996;22(3):446-55
- 5. Casao MA; Leiva J; Diaz R; Gamazo C. Anti-phosphatidylcholine antibodies in patients with brucellosis. J Med Microbiol 1998; 47(1):49-54.

2025-08-08



## **INTENDED USE**

The Calbiotech Brucella IgG ELISA Kit is intended for the detection of IgG antibody to Brucella in human serum or plasma. For Research Use Only. For professional use only. Not for use in diagnostic procedures.

## SUMMARY AND EXPLANATION

Brucella species are intracellular gram-negative bacteria transmitted via direct contact with infected animals or consumption of contaminated animal products. IgG antibodies develop during or after the acute phase of infection and can persist for extended periods. IgG detection supports research on exposure history, population seroprevalence in endemic areas, and long-term immune response durability.

## PRINCIPLE OF THE TEST

Diluted patient serum is added to wells coated with purified antigen. IgG specific antibody, if present, binds to the antigen. All unbound materials are washed away, and the enzyme conjugate is added to bind to the antibody-antigen complex, if present. Excess enzyme conjugate is washed off and substrate is added. The plate is incubated to allow the hydrolysis of the substrate by the enzyme. The intensity of the color generated is proportional to the amount of IgG specific antibody in the sample.

	MATERIALS PROVIDED	96 TESTS
1.	Microwells coated with Brucella abortus antigen	12x8x1
2.	Sample Diluent: 1 bottle (ready to use)	22 mL
3.	Calibrator: 1 Vial (ready to use)	1 mL
4.	Positive Control: 1 vial (ready to use)	1 mL
5.	Negative Control: 1 vial (ready to use)	1 mL
6.	Enzyme conjugate: 1 bottle (ready to use)	12 mL
7.	TMB Substrate: 1 bottle (ready to use)	12 mL
8.	Stop Solution: 1 bottle (ready to use)	12 mL
9.	Wash Concentrate 20X: 1 bottle	25 mL

## MATERIALS NOT PROVIDED

- Distilled or deionized water
- Precision pipettes
- 3. Disposable pipette tips
- 4. ELISA reader capable of reading absorbance at 450nm
- Absorbance paper or paper towel
- 6. Graph paper

## STORAGE AND STABILITY

- 1. Store the kit at 2-8° C.
- Keep microwells sealed in a dry bag with desiccants.
- 3. The reagents are stable until expiration of the kit.
- 4. Do not expose test reagents to heat, sun or strong light.



#### WARNINGS AND PRECAUTIONS

Potential biohazardous materials:

- The standards contain human source components which have been tested and found non-reactive for hepatitis B surface antigen as well as HIV antibody with FDA licensed reagents.
  However, as there is no test method that can offer complete assurance that HIV, Hepatitis B virus or other infectious agents are absent, these reagents should be handled at the Biosafety Level 2, as recommended in the Centers for Disease Control/National Institutes of Health manual, "Biosafety in Microbiological and Biomedical Laboratories." 1984.
- Do not pipette by mouth. Do not smoke, eat, or drink in the areas in which specimens or kit reagents are handled.
- The components in this kit are intended for use as an integral unit. The components of different lots should not be mixed.
- 4. It is recommended that standards, control and serum samples be run in duplicate
- Optimal results will be obtained by strict adherence to this protocol. Accurate and precise pipetting, as well as following the exact time and temperature requirements prescribed are essential. Any deviation from this may yield invalid data.

## SPECIMEN COLLECTION AND HANDLING

This assay is designed for use with human serum or plasma samples obtained in compliance with applicable laws, regulations, and institutional policies. Handle and store samples using procedures appropriate for research use. Samples may be stored refrigerated (2–8 °C) for up to seven days, or frozen (–20 °C or below) for up to six months. Avoid repetitive freeze–thaw cycles.

## REAGENT PREPARATION

Prepare 1X Wash buffer by adding the contents of the bottle (25 ml, 20X) to 475 ml of distilled or deionized water. Store at room temperature (20-25 °C).

## **ASSAY PROCEDURE**

Prior to assay, allow reagents ti stand at room temperature. Gently mix all reagents before use.

- 1. Bring all specimens and kit reagents to room temperature (20-25 °C) and gently mix.
- 2. Place the desired number of coated strips into the holder.
- 3. Negative control, positive control, and calibrator are ready to use. Prepare 1:21 dilution of test samples, by adding 10 ul f the sample to 200 ul f sample diluent. Mix well.
- 4. Dispense 100 μl f diluted sera, calibrator and controls into the appropriate wells. For the reagent blank, dispense 100μl ample diluent in 1A well position. Tap the holder to remove air bubbles from the liquid and mix well. Incubate for 20 minutes at room temperature.
- 5. Remove liquid from all wells. Wash wells three times with  $300\mu l$  of 1X wash buffer. Blot on absorbance paper or paper towel.
- 6. Dispense 100  $\mu$ l of enzyme conjugate to each well and incubate for 20 minutes at room temperature.
- 7. Remove enzyme conjugate from all wells. Wash wells three times with 300  $\mu$ l of 1X wash buffer. Blot on absorbance paper or paper towel.
- 8. Dispense 100 µl of TMB substrate and incubate for 10 minutes at room temperature.
- 9. Add 100  $\mu$ l of stop solution.
- Read O.D. at 450 nm using ELISA reader within 15 min. A dual wavelength is recommended with reference filter of 600-650 nm.

# **CALCULATION OF RESULTS**

- Check Calibrator Factor (CF) value on the calibrator bottle. This value might vary from lot to lot. Make sure you check the value on every kit.
- 2. Calculate the cut-off value: Calibrator OD x Calibrator Factor (CF).
- Calculate the Ab (Antibody) Index of each determination by dividing the O.D. value of each sample by cut-off value.

## LIMITATIONS OF THE TEST

1. Lipemic or hemolyzed samples may cause erroneous results.